

## Brief Instructions for wipe- and pure substance testings

### 1. Sample Collection

Wipe over the surface with the **dry** collector for approx. 10 seconds.

While wiping roll the collector so that the complete area of the mixro fibre is used on all sides.

When testing pure substances: **Do not** dip the collector totally into the pure substance, only wipe on the surface of the surroundings of the substance (f.e. Inner surface of the packing or bottle).



### 2. Wash out sample

Put the collection stick into the buffer bottle.

Agitate it by rotary movements for **10 seconds**.



Put the collector back in its tube. Close the bottle with the dropper lid.



### 3. Put sample to test

Pipet **7 drops** in each well of the incubation device.

Close the lid up to position 1 (see stopper).




Shake the test for **10 seconds**



### 4. Start test

**Test has to lay flat now!**

Bend the stopper and press down the lid completely.

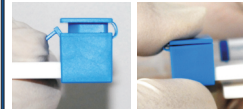

### 5. Reading of test results

(The intensity of the lines may vary).

As soon as **all lines** have formed the test is negative.

In case one or more lines are not visible, read the result in the **6<sup>th</sup> minute**.

**Do not read the result after 6 minutes!**

When performing saliva testings use the saliva test instructions !

