

Brief Instructions for saliva testings


1. Increase salivation of donor

Rub over the aroma field for **4 sec.**



2. Sample collection

Collect saliva by rotary movements between cheek and gum from both sides of the mouth. (30 sec.)

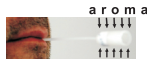

3. Wash out sample

In the buffer bottle agitate the collector quickly back and forth for **10 sec.**



Put the collector back in the tube. Close the bottle with the dropper lid.



4. Bring sample to the test

Pipet **7 drops** in each well of the blue device.




Close the lid until position 1 (see stopper).



5. Incubation phase



Shake the test for **10 seconds**

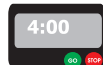

Wait 4 minutes

6. Start test

Test has to lay flat now!
Bend the stopper and press down the lid completely.

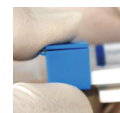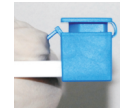

7. Reading of test results

(The intensity of the lines may vary).

As soon as all lines have formed the test is negative.

In case one or more lines are not visible, read the result **in the 8th minute.**

Do not read the results after 8 minutes!

