

## 1. Switching on / off

The on/off switch is located on the upper, right side of the UMPC. Briefly push it to the side to switch the unit on/off. Click on the green ball and the MOBILE READER program




## 2. Performing a new measurement

Click on "Run".



## 3. Camera sequence starts

After a few seconds the preview of the camera appears. Verify whether a test has been inserted. Then click on continue. The camera starts the measurement.

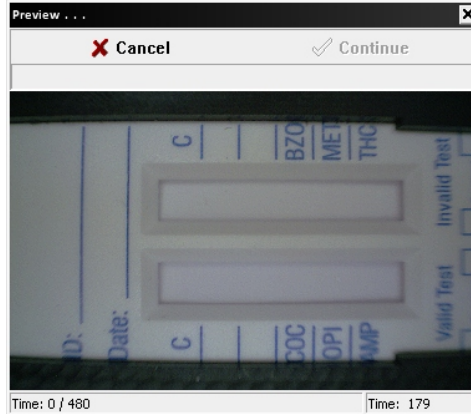


A signal tone appears when the measurement is completed. The information about the availability of the result appears on the screen.



## 4. Measurement results

The result is displayed immediately as **NEGATIVE** or **POSITIVE**. It is also validated if the test was valid or not valid. A photo of the measured test cassette is displayed on the screen also.

- sample ID
- drug parameter
- name of operator
- signature and comment field

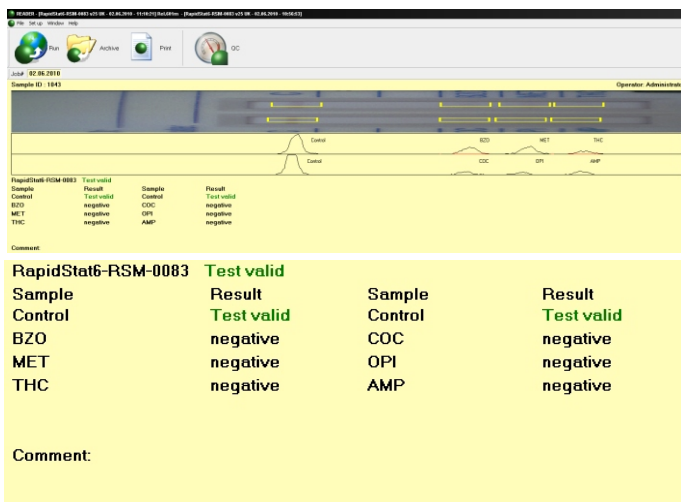


Sample	Result	Sample	Result
Control	Test valid	Control	Test valid
BZO	negative	COC	negative
MET	negative	OPI	negative
THC	negative	AMP	negative

Comment:

## 5. Printout

Click on **Print** to print out the result.

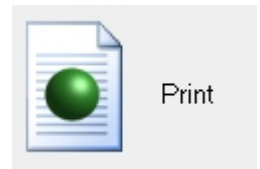

Assay Report			
RapidStat6-RSM-0083			
<b>Administered by:</b> MAVAND Solutions GmbH Ulrichstr. 23 Mössingen B-W 72116 Deutschland		<b>Sample ID</b> Test	15.06.2010 1064
		<b>Date</b>	15.06.2010 - 16:56:12
Sample	Result	Sample	Result
Control	Test valid	Control	Test valid
BZO	negative	COC	negative
MET	negative	OPI	negative
THC	negative	AMP	negative